4 LEGS CANE - BLACK	K10066
4 LEGS CANE - BLUE	K10070
4 LEGS CANE - RED	K10071

The **4 Legs Cane** has been designed for people who need extra support when walking. Its 4 tips include 2 wide and 2 narrow tips to increase stability by adapting to all surfaces. It has a secure grip handle that adapts to the shape of the hand.

HEIGHT ADJUSTMENT

1. Stand upright on a firm, flat surface, allowing your arm to hang naturally at your side.
2. Join the two tubes together and slide the aluminium tube until the crutch is at the correct height.
3. Remove the height adjustment clip and slide the aluminium tube until the cane is at the correct height.
4. Rotate the handle tube depending on the arm on which it is to be used, always leaving the adjustments on the side of the wide end caps.
5. The top of the handle should be at the level of the inside of the wrist.
6. Replace the clip.

FEATURES

- Approximate weight of the pole is 3 lbs.
- PA6 + 50GF (Base). Steel (Lower tube). Aluminium (Upper tube). Rubber ferrules and grip.

WARNINGS

- Do not use this product without reading and understanding the instructions. If necessary, contact a health care professional, dealer or technical personnel before using this product to avoid possible injury.
- The KMINA 4 Legs Cane provides support, increases stability and aids walking.
- Provides ambulatory assistance for individuals weighing up to 350 lbs.
- Before using the cane, ensure that the height adjustment clip is securely attached to avoid possible injury or damage.
- Frequently inspect the pole tip or cleat for damage or wear.
- Replace any broken, damaged or worn parts.
- Consult your doctor or therapist at all times to determine proper adjustments and use of the cane.
- Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Keep your cane clean using a damp cloth. Do not use abrasive cleaning materials.

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