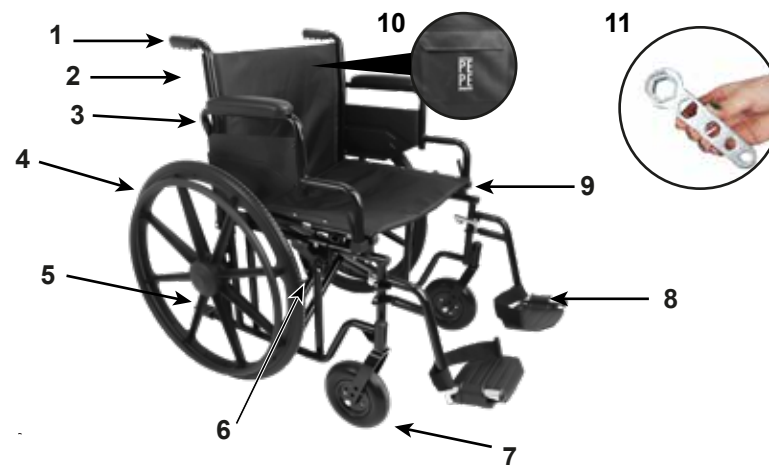
**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE**

**PEPE - BARIATRIC WHEELCHAIR
PEPE - SILLA DE RUEDAS BARIÁTRICA
PEPE - BARIATRISCHER ROLLSTUHL
PEPE - FAUTEUIL ROULANT BARIATRIQUE
PEPE - CARROZZINA BARIATRICA**

1 COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI

| | ENGLISH | ESPAÑOL | DEUTSCH | FRANÇAIS | ITALIANO |
|------------|--------------|------------------|------------------|---------------------|--------------------|
| 1. | Handle | Empuñadura | Handgriff | Poignée | Maniglia |
| 2. | Backrest | Respaldo | Rückenlehne | Dossier | Schienale |
| 3. | Armrest | Reposabrazos | Armlehne | Accoudoir | Bracciolo |
| 4. | Rear wheel | Rueda trasera | Hinterrad | Roue arrière | Ruota posteriore |
| 5. | Assist lever | Palanca de ayuda | Assistenzhebel | Levier d'assistance | Leva di assistenza |
| 6. | Brake lever | Palanca de freno | Bremshebel | Levier de frein | Leva del freno |
| 7. | Front castor | Rueda delantera | Vorderrad | Roue avant | Ruota anteriore |
| 8. | Footrest | Reposapiés | Fußstütze | Repose-pieds | Poggiapiedi |
| 9. | Seat | Asiento | Sitz | Siège | Sedile |
| 10. | Pocket | Bolsillo | Tasche | Poche | Tasca |
| 11. | Fixed key | Llave fija | Fester Schlüssel | Clé fixe | Chiave fissa |

2

| WIDTH | HEIGHT | LENGTH | FOLDING DIMENSIONS |
|---------------|---------------|--------------|-----------------------------|
| 80 cm (31.5") | 95 cm (37.5") | 115 cm (45") | 85x95x34 cm (33"x37.5"x13") |
| | | | |

| SEAT HEIGHT | SEAT DIMENSIONS | BACKREST HEIGHT | Ø REAR WHEEL | Ø FRONT CASTOR |
|-------------|------------------------|-----------------|---------------|----------------|
| 48 cm (19") | 55x45 cm (21.5"x17.5") | 45 cm (17.5") | 65 cm (25.5") | 19 cm (7.5") |

| MAX. USER WEIGHT | NET WEIGHT | TOTAL WEIGHT (with footrest) |
|------------------|---------------|------------------------------|
| 225 kg (500 lb) | 23 kg (50 lb) | 24,5 kg (54 lb) |

EN The **PEPE - Bariatric Wheelchair** has been designed to be self-propelled by the user using the large diameter rear wheels, or by another person using the handles. It can support a maximum weight of 500 lbs and is designed for independent patients, elderly people, people with mobility problems or who have some physical impediment but who have sufficient autonomy to be able to self-propel themselves safely.

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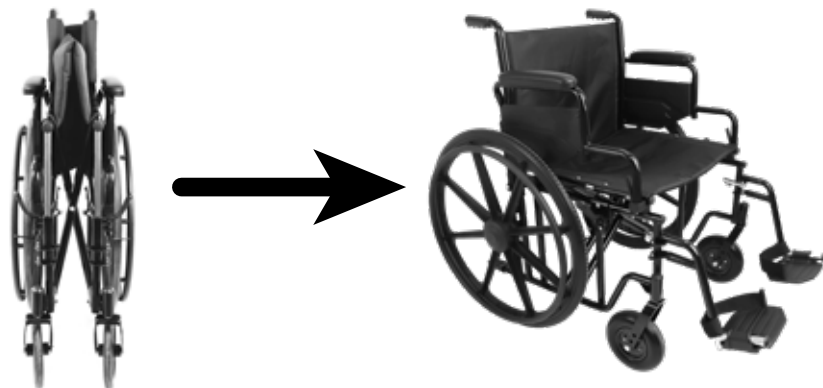
3 ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO

1

EN 1. **Unfold** the wheelchair by pulling the armrests outward.

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2

EN 2. **Insert the footrests** by fitting the holes and rotate it until the footrest is facing the front.

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1

EN 1. **To remove the footrests**, use the metal lever to unclip the footrests and remove them from the chair frame.

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MATERIALS

- Structure and footrest: steel.
- Armrests, handles and plastic finishes: PVC.
- Fabric: fireproof nylon.
- Wheels and castors: PU.

WARNINGS

- Do not use the wheelchair without carefully reading the instructions.
- This chair may not be used as a seat in vehicles of any kind.
- Always transfer the wheelchair user to a seat approved by the vehicle manufacturer and use the available safety devices.
- Do not use this wheelchair on streets, roads or any surface with vehicle traffic.
- Do not attempt to use the wheelchair on ramps without proper assistance.
- Before inserting or removing the user, the wheelchair must be on a smooth and stable surface.
- Do not apply weight on the footrests when inserting or lifting the user into or out of the wheelchair.
- Do not lean or drop weight on the top of the wheelchair backrest, it may cause the wheelchair to tip over.
- Do not stand on the wheelchair.
- Sudden changes in direction when using a ramp or slope can cause instability and/or the possibility of tipping.
- Keep in mind that this chair is intended for transporting another person, and not for personal transport.

- Never stretch so far that you have to move around in the seat or out of the chair.
- When you have to reach for objects that require you to lean, make sure the front wheels point forward, as this will stabilize the chair.
- The temperature of the wheelchair may increase when exposed for long periods of time to external heat sources, such as sunlight.
- If you will be storing the wheelchair for an extended period of time, do not leave it exposed to prolonged weathering.
- Do not exceed the maximum supported weight, which is 500 lbs.
- In accordance with the manufacturer's recommendations, this product is suitable for recycling at designated recycling points.
- Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Retain these instructions for future reference.

MAINTENANCE

- Inspect the chair before each use to ensure that there is no damage or excessive wear. If you find any signs of wear or damage to your chair, DO NOT USE IT.
- Regularly check that all parts are sound before operation and that the casters turn smoothly. If you notice that the casters are loose, tighten them using the wrench provided.
- Keep the wheelchair clean, the steel tubes can be easily cleaned with a damp cloth. The fabric parts should be washed by hand (max. 30°) and with neutral soap.

WARRANTY

- KMINA warrants this wheelchair to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement, such as brakes, wheels and handgrips. This warranty does not cover:
- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
 - Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
 - KMINA is not liable for accidental damage or damage to property.

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