

TENA Slip (All-in-one Pads)

Breathability for healthy skin

TENA Slip with ConfioAir™ is an all-in-one incontinence product, which helps to maintain healthy skin, providing breathability and leakage protection. ConfioAir™ allows skin to breathe and the new breathable backsheet and sides help maintain dryness for comfort and healthy skin.2 TENA Slip now features new wide hook tapes which are easy to fasten and allow multiple adjustments to achieve a perfect fit. The fast-acting, highly absorbent core securely protects against leakage for comfort and dignity.

TENA Slip Fitting Guide – Lying and Standing



With the user lying on their side, slide the pad between the legs from front to back.



Smooth out the pad.



Fit both the bottom hook tapes, one at a time, angled slightly upwards and then fit both top hook tapes, one at a time angled slightly downwards. Check for comfort and fit. The pad allows for multiple refastening and adjustment of the hook tapes.



Gently press and smooth the hook tapes on to the textile-like back-sheet.



Give a gentle pull to anchor the hook tapes into the fibres of the back-sheet. This ensures the secure fixation.



With the user standing, place the pad between the legs from front to back.




Fit both the bottom hook tapes, one at a time, angled slightly upwards.



Then fit both top tapes, one at a time angled slightly downwards. Check for comfort and fit. The pad allows for multiple refastening and adjustment of the hook tapes.



Gently press and smooth the hook tapes on to the textile-like back-sheet.

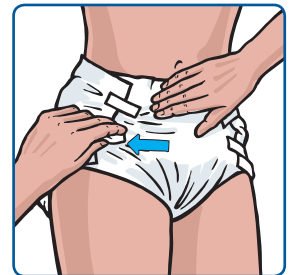

Give a gentle pull to anchor the hook tapes into the fibres of the back-sheet. This ensures the secure fixation.

TENA Slip dos and don'ts

- ✓ Measure around the hips for correct size
- ✓ Assess for correct absorbency
- ✓ Fold pad lengthways to create bowl shape
- ✓ Fit from front to back
- ✓ Seal bottom tabs pointing upwards and top tabs pointing downwards
- ✓ Ensure standing gathers around leg openings are pulled up before fitting
- ✓ Ensure a close fit is achieved
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, used sparingly to ensure the product works effectively
- ✗ Never 'twist' the product
- ✗ Never shake the product in order to open