


## WARRANTY

Model: .....

Serial number: .....

Stamp of the point of sale  
and legible signature of the Seller: .....

- 1) ANTAR General Partnership, 03-068 Warsaw, Zawiślańska Street, No. 43 hereby grants a 12 month warranty on the wheelchair from the date of the purchase of the product by the Buyer.
- 2) During the warranty period, ANTAR Company undertakes to make the necessary repairs, in order to restore the proper functioning of the system, within 14 days of receipt of the product by the service, along with the proof of purchase and the warranty card.
- 3) ANTAR reserves the right to replace the product if it considers that the repair costs make it unviable. Repair or replacement of defective products does not extend the warranty period.
- 4) If it is determined that the product functions improperly, it shall be immediately send to the ANTAR's address (shown in point 1) or contact the point of purchase.
- 5) Before operating, please read the instructions and follow them closely. Use of the product contrary to its purpose and recommendations causes irreversible loss of warranty.
- 6) The warranty does not cover damages caused by piercing or cutting the product with the use of a sharp object; damages due to the influence of external mechanical forces; pressure exceeding the maximum permissible weight of the user, as described in the manual; contact the product with alcohol, fatty and oily substances or gasoline. Warranty does not cover parts subject to normal wear during use, such as: tires, soft fabrics, armrests, bearings, rubber elements and others.
- 7) The product being sent to the service as part of the complaint cannot cause epidemics.
- 8) THIS WARRANTY IS VALID ONLY WITH THE PROOF OF PURCHASE (RECEIPT, BILL, INVOICE). PLEASE ATTACH THE PROOF OF PURCHASE IN CASE OF THE SUBMISSION OF ANY CLAIM. FOR RECOGNITION OF EXTENDED PERIOD OF WARRANTY FOR PRODUCTS, WHICH PURCHASE WAS REIMBURSED BY THE NATIONAL HEALTH FUND, PLEASE ATTACH THE COPY OF 'ORDERS FOR MEDICAL SUPPLIES BEING ORTHOPEDIC ITEMS AND AUXILIARIES'.
- 9) Claims submitted without the proof of purchase and the warranty card along with the serial number of the product will not be considered.

 Manufacturer:  
ANTAR Sp. J.  
03-068 Warszawa, ul. Zawiślańska 43, Poland  
Tel. +48 22 518 36 00  
www.antar.net e-mail:antar@antar.net



# OPERATING MANUAL AT51031 ROLLATOR WALKER



Outdoor rollator walker is equipped with parking brake and the seat with backrest with upholstery. The height of the handle is adjustable and rollator walker can be folded for transport or storage.

Before folding or using rollator walker for the first time it is necessary to read carefully operating manual and it should be kept in safe place. Before exact reading this operating manual the device can NOT be folded or used.

#### APPLICATION

The device is used as walking aid, support and help to carry smaller objects as well inside as outside. It is designed for the use by the user with reduced mobility and below mentioned maximal body weight

In order to guarantee the safety of the user of rollator walker, the user should have proper physical, cognitive and visual abilities. When in doubt, please contact the seller/specialist who recommended using rollator walker.

It is very important that the user acquaint himself/herself safety hints in this operating manual exactlt before using the device for the first time. Hints mentioned here should prevent possible injury of the user or third party and also the damage of rollator walker or the objects in rollator walker.

#### HINT:

It is not allowed to stand on particular parts of the frame. The frame of the device is not designed for such purpose.

#### SAFETY

Rollator walker can be used by the person with maximal weight of 135 kg.

#### IT HAS TO BE OBSERVED:

- (1) Make sure that the brake of unattended rollator walker is always locked / engaged.
- (2) Before the beginning of the use, please make sure that all wheels, including moving wheels, are securely fastened.
- (3) Make sure that brakes function correctly.
- (4) Before the first use, the height of the handle and its correct fastening should be checked.
- (5) Make sure that rollator walker is fully unfolded.
- (6) During unfolding rollator walker, the frame should be set in correct position.
- (7) To avoid squeezing fingers, stay away from folding mechanism.
- (8) In order to adapt rollator walker properly to your needs, seek advice of qualified personnel.
- (9) Caution should be kept during lifting or carrying rollator walker.

#### IT IS FORBIDDEN:

- (1) Using rollator walker for the transport of people – The device is not designed for the use as the wheelchair.
- (2) Do not sit on rollator walker without locking the brake - it may put rollator walker in motion and the injury of the user or third party can be caused.
- (3) Do not sit on rollator walker on the slope without activated parking brake.
- (4) Do not sit on the seat with the swing - sit carefully and in controlled way, ask for help, if necessary.
- (5) Do not hang any objects on brake lever or on handles; it may disturb functioning the brakes and/or the stability of rollator walker.
- (6) Do not stand on rollator walker and do not use it as the footrest.

#### ASSEMBLY MANUAL

Make sure that rollator walker is not damaged during transport. In the case of visible damage, do not use rollator walker. In this case, please contact the seller to have rollator walker repaired or to return it.



- A – Backrest
- B – Brake lever
- C – Handles with the adjustment of the height
- D – The seat
- E – Bag
- F – Diameter of rear wheels 30 cm
- G – Diameter of front wheels 30 cm

1. Remove the packaging. Rollator walker is folded.
  2. Unfold rollator walker, push the seat down on both sides - Figure 4.
- HINT: Both sides have to be locked in correct setting with audible “click”.
3. Hang the bag.



figure 4

#### THE ADJUSTMENT OF THE HEIGHT

##### Proper adaptation

Before the first use of rollator walker make sure that it is fully unfolded and that the frame is properly locked and rigid. It is important to set the height of rollator walker properly in comfortable position..

1. For this purpose lower hands and set handles of rollator walker at the height of wrists.
2. During holding handles, elbows should be bent at an angle of 15°-20°.
3. If you have to bow or raise arms to reach rollator walker, adjust the height.
4. Unscrew wing screw simultaneously and set the handle at correct height. Repeat this action on the other side of rollator walker and check before the use, if both handles are locked at the same height and correctly.



HINT: To determine correct height of rollator walker, please contact the doctor or professional seller.

#### SETTING AND OPERATING THE BRAKES

Service brake (during pushing): Pull brake lever upwards.

Parking brake (during parking): Push brake lever downwards. (see figure 5).



figure 5

Brake setting: To increase the tension of brake cable, turn the screw counterclockwise; then - to keep such setting - turn fastening button clockwise and hold adjustment screw simultaneously (see figure 6).

HINT: In the case of any uncertainty, contact Customer Service. Changes of brake settings can lead to the reduction of braking force.



figure 6

#### FOLDING AND STORING ROLLATOR WALKER

1. Pull the strap in the middle of the seat upwards to fold rollator walker - Figure 7.

#### MAINTENANCE

1. Make sure that ALL elements of equipment are always correctly tightened.
2. Check, if screws, buttons and knobs of wheels and moving wheels are tightened. Check, if the brake functions properly and that handles are correctly secured.
3. Worn or damaged parts should be immediately replaced.

#### CLEANING

You need damp cloth or sponge and warm water with soap to clean rollator walker. If necessary, also conventional disinfectants can be used. Make sure that there is no water or remains of cleaning agent on clean surfaces and that they are not slippery. Do not use rough cleaning agents or scrapers - they may damage paint film.



figure 7